Data-Cyte® Plus Reagent Red Blood Cells 3±1%

U.S. Licence No. 1740

*For use in the identification of unexpected antibodies**For in vitro diagnostic use*

IVD

Summary and Principle

Careful and complete identification of an unexpected antibody is important in the diagnosis and treatment of hemolytic disease of the newborn (HDN) and certain blood dyscrasias, as well as in the prevention of transfusion reactions due to infusion of incompatible Red Blood Cells. Most clinically significant antibodies can be identified by agglutination in routine procedures using Reagent Red Blood Cells of known antigenic constitution.^{1,2}

Data-Cyte® Plus Reagent Red Blood Cells is a panel of suspensions of group O Red Blood Cells from 11 individual donors. These donor Red Blood Cells differ in antigenic configuration and are selected to enable identification of most single antibodies, as well as a majority of frequently found combinations of antibodies. The presence or absence of antigens of each of the major blood group systems is indicated for each of the 11 Red Blood Cells on the antigenic constitution matrix accompanying the product. Data-Cyte® Plus may be utilized in the commonly accepted antibody identification techniques.

Antibodies react with Red Blood Cells possessing the corresponding antigenic determinants. These antibodies may agglutinate Red Blood Cells in saline, low ionic strength solution (LISS), high protein media, or antiglobulin testing. Following this principle, an antibody may be identified by its pattern of reactivity with a panel of human Red Blood Cells whose antigenic constitution is known.

Reagent

Data-Cyte® Plus Reagent Red Blood Cells: a panel of 11 individual group O human Red Blood Cells samples, 3±1% suspensions in isotonic medium with added buffers (bicarbonate and phosphate) and preservatives (0.03% neomycin and 0.05% chloramphenicol). The suspending medium contains no ingredients to inhibit complement mediated hemolysis. Frozen/thawed Red Blood Cells may have been used in this product. No U.S. standard of potency. Meets other FDA requirements.

Store at 2–8°C. **Do not freeze.** If stored appropriately at 2–8°C the product is stable after the first opening until the indicated expiration date. Resuspend by gentle inversion immediately prior to use. Reagent Red Blood Cells are ready to use; washing Reagent Red Blood Cells before use is optional. Indication of deterioration: notable hemolysis (which may be caused by microbial contamination or improper handling), darkening of Reagent Red Blood Cells or spontaneous clumping. The reactivity of the product may diminish during the dating period.

Caution: All blood products should be treated as potentially infectious. Source material from which this product was derived was found negative when tested in accordance with current FDA required tests. No known test methods can offer assurance that products derived from human blood will not transmit infectious agents.

The pipette of the vial contains natural rubber latex which may cause allergic reactions.

Specimen Collection and Preparation

No special preparation of the patient is required prior to specimen collection. Serum from freshly clotted blood is preferred. For optimum test results, serum should be stored at 2–8°C no longer than 48 hours prior to testing; however, serum may be frozen at -20 to -80°C and tested at a later time if necessary. Plasma samples may be used, however, use of plasma may result in failure to detect complement dependent antibodies due to its low complement activity.^{1,2}

Procedure

Reagent Provided

Data-Cyte® Plus Reagent Red Blood Cells, 3±1%, panel of 11 Red Blood Cells samples.

Materials Required but Not Provided

1. Disposable test tubes (12 x 75 mm or 10 x 75 mm)
2. Physiologic saline
3. Anti-human globulin
4. Potentiator
5. Enzyme reagent required for Saline/Enzyme Procedure only
6. Coombs control cells
7. Control serum for quality control
8. Optical aid³
9. Centrifuge (calibrated for 1000 rcf* or 150 rcf*)
10. Waterbath or heating block at 37°C
11. Timer
12. Pipettes

Procedure Outline

The Data-Cyte® Plus panel is uniquely designed so that it may be used either independently or in conjunction with reagent antibody screening cells. When combined with the results from screening cells and auto-control, only the first four Red Blood Cells of Data-Cyte® Plus panel need be used to provide preliminary identification of the most common Red Blood Cell antibodies. If the antibody cannot be clearly identified using this "mini-panel," the remaining Red Blood Cells of the panel and selected additional Red Blood Cells (if required) may be used to complete the identification.

Note: If a potentiator is to be used, follow manufacturer's instructions for preparation and testing of samples instead of the procedure suggested below.

1. Place 2 or more drops of the patient or donor serum in a tube for each Reagent Red Blood Cells to be tested.
2. Add 1 drop of each Data-Cyte® Plus Reagent Red Blood Cells suspension to the appropriate tube.
3. Shake all tubes to mix reagents.
4. If immediate spin testing is desired, centrifuge for 15–20 seconds at approximately 1000 rcf* (1 minute at approximately 150 rcf*) or time appropriate to the calibration of the centrifuge.
5. Gently resuspend Red Blood Cells completely and examine immediately for agglutination or hemolysis.³ Grade and record results.

Indirect Antiglobulin Test

6. Incubate at 37°C for 15–60 minutes or according to directions for the potentiator used.
7. Immediately centrifuge, examine and interpret as in Steps 4 and 5. Record results 37°C testing.
8. Fill each tube with physiologic saline added in a forceful stream. Centrifuge to pack Red Blood Cells. Carefully decant supernatant. Shake to resuspend Red Blood Cells.
9. Repeat Step 8 twice for a total of 3 washings.
10. Add anti-human globulin to each tube according to manufacturer's instructions and shake to mix.
11. Centrifuge, examine and interpret as in Step 4 and 5. Grade and record results.
12. Negative reactions obtained with anti-human globulin should be tested by adding 1 drop of IgG-sensitized cells.

Note: The reactions should be interpreted immediately after centrifugation due to the possibility of dissociation of the antigen-antibody complex.

Saline Room Temperature – Low Temperature

6. Incubate at room temperature (20–25°C) for 15–30 minutes.
7. Centrifuge, examine and interpret as in Steps 4 and 5. Grade and record results.
8. If detection of cold agglutinins is desired, incubate tubes at 5°C for 5–15 minutes. Then centrifuge, examine and interpret as in Step 4 and 5. Grade and record results of low temperature testing.

*rcf = 0.00001118 x rotating radius (cm) x rpm²

Quality Control

Depending on national regulations for quality control

1. a known relatively weak IgG reactive antibody should be tested with a different Data-Cyte® Plus suspension each day the Reagent Red Blood Cells are in use to assure continuous reactivity of the Reagent Red Blood Cells. This testing may be conveniently performed using Blood Bank Quality Assurance (BBQAll) Testing Reagents, available from Medion Diagnostics AG or
2. (common practice in Europe) a control reagent containing a blend of weak Rh antibodies should be run in parallel with each series of tests (Sero-Control is available in Europe from Medion Diagnostics AG).

Use of an autocontrol is recommended to help distinguish between autoantibodies and alloantibodies.²

To ensure proper centrifugation, each individual centrifuge should be calibrated for the specific test procedure being performed. Red Blood Cells should be packed firmly, but negative control Red Blood Cells should resuspend easily.⁴

Results

Agglutination and/or hemolysis (positive reaction) in one or more Data-Cyte® Plus tubes at any phase of the test procedure prior to the addition of Coombs control cells indicates the presence of unexpected antibodies. Such antibodies are usually directed against the known antigens present on the panel Red Blood Cells, but may be directed against an antigen not indicated on the antigenic constitution matrix.

The lack of both agglutination and hemolysis (negative reaction) in the test procedure indicates the absence of antibodies to antigens contained on the Reagent Red Blood Cells.

Interpretation

Identification of the antibody(ies) present may be conveniently performed by the "crossing out" method using the antigenic constitution matrix accompanying the lot of Data-Cyte® Plus Reagent Red Blood Cells.

1. Choose the first Red Blood Cells giving a negative reaction at all phases of testing. Cross out all antigenic determinants present on that Red Blood Cells.
2. Repeat Step 1 for all other negative Red Blood Cells.
3. Circle remaining antigens.
 - a. If only one antigen is circled, check to see that all Red Blood Cells which reacted possess the antigen. If so, the antibody is probably directed against that antigen and can be identified as such.
 - b. If several antigens are circled, check to see if any one of those antigens is present on all the reacting Red Blood Cells. If so, additional Red Blood Cells lacking that antigen, but possessing the others circled, should be tested to determine if multiple antibodies are present.
 - c. Antigen typings on patient/donor Red Blood Cells may be useful to rule out antibodies.
 - d. If high incidence antibodies or multiple antibodies are present, all Red Blood Cells may be agglutinated. A reference laboratory should be consulted if rare Red Blood Cells are not available for testing.

If the autocontrol is positive, the serum may contain autoantibody and further testing may be indicated.²

Limitations of Procedure

A 15-minute incubation at 37°C may not be adequate to detect some weak blood group antibodies if no potentiating medium is added to the test system. Red Blood Cells having low frequency antigens, a double dose of antigen may be required to detect very weakly reacting antibodies; therefore, negative reactions with panel Red Blood Cells do not always indicate absence of unexpected antibodies in the serum under test. Because of the high incidence of the Fy^a gene in the black population, it cannot be assumed that the phenotypes Fy (a+b-) and Fy (a-b+) in black donors represent homozygous expressions of the Fy^a or Fy^b genes⁵. If antibodies to high incidence antigens or multiple antibodies are present, all Red Blood Cells may be agglutinated.

As in all serological tests, such factors as contaminated materials, improper incubation time or temperature, improper centrifugation or improper examination for agglutination may give rise to false test results.

False negative results may occur if

1. Red Blood Cells are not properly washed or human globulins are present as contaminants in glassware. These residual globulins will neutralize the globulin-reactive antibodies present in the anti-human globulin.
2. antibody elutes from Red Blood Cells during incubation or washing.
3. Red Blood Cells and/or serum are stored improperly and lose reactivity.
4. anti-human globulin is inadvertently omitted.
5. Red Blood Cells are improperly centrifuged.
6. incubation times and/or temperatures are incorrect for proper Red Blood Cells sensitization.
7. resuspension technique is too vigorous to preserve agglutination of weakly sensitized Red Blood Cells.

False positive results may occur if

1. Red Blood Cells have microbial contamination.
2. Red Blood Cells are improperly centrifuged.
3. antibodies to antibiotics or to other ingredients in the Red Blood Cells suspending medium or in the potentiators used are present in the test serum.
4. incomplete resuspension may counterfeit agglutination.
5. in rare cases, the test serum contains an antibody directed at one of the components of the reagent diluent.

Specific Performance Characteristics

Each lot of Data-Cyte® Plus Reagent Red Blood Cells is carefully prepared to permit detection of antibodies to the selected Red Blood Cells antigens when used as outlined in these procedures.

All antigen typings listed on the antigenic constitution matrix are confirmed using two sources of antiserum except for the following which, due to the rarity of the antibodies, may be tested with only one source if a second source is unavailable: f, V, Lu^a, Js^a, Js^b, Xg^a, Vel, Ge, Yt^a, Di^a, Di^b, and special typings (other antigens).

Unless otherwise indicated, the Red Blood Cells of Data-Cyte® Plus donors have been phenotyped as follows:

Positive: H, I, U, Kp^b, Js^b, Vel, Ge, Yt^a, Di^b

Negative: M^a, Vw, W^a, Di^a

Identified low incidence antigens present are indicated on the antigenic constitution matrix. Direct antiglobulin tests are negative on all Red Blood Cells.

The stability of the product is monitored throughout the dating period.

As with all Red Blood Cells, the reactivity of the product may decrease during the dating period. The rate at which antigen reactivity is lost is partially dependent upon individual donor characteristics that are neither controlled nor predictable by the manufacturer. However, if properly stored when not in use, the reagent can be expected to perform as described throughout its dating.

Bibliography

1. Mollison P.L., Blood transfusion in clinical medicine. 10th Edition: Blackwell Scientific Publications; 1997: Chapter 8.
2. Technical Manual of the American Association of Blood Banks. 14th Edition; 2002, Chapter 19 and 20.
3. Ibidem: Chapter 18, p. 383.
4. Ibidem: Method 8.5, p.765.
5. Ibidem: Chapter 15, p. 325.

Warranty

This product is warranted to perform as described in its labeling and in the product literature and Medion Diagnostics AG disclaims any implied warranty of merchantability or fitness for any other purpose, and in no event shall Medion Diagnostics AG be liable for any consequential damages arising out of the aforesaid express warranty.

Manufacturer

Manufactured by:
Medion Diagnostics AG
Bonnstrasse 9, CH-3186 Düringen/Switzerland
Made in Switzerland

© Copyright 2006, Medion Diagnostics AG
All rights reserved.

Distributed in US by:
Olympus America Inc.
3500 Corporate Parkway
Center Valley, PA, 18034-0610
Customer Service 800 223 0125
Technical Service 800 447 5852

Distributed in US by:
Medion Diagnostics International Corporation
8833 NW 68th Street
Miami, FL 33166, USA
Tel (305) 362 8595
Fax (305) 362 1796

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